

# Ipso Norge AS.

Trollåsveien 6, 1414 TROLLÅSEN

Tel. 66 99 50 90 - E-mail: post@ipso.no – www.ipso.no

## Standard operating conditions: DVGW03-MULTIW-LE-537F1-MIX

Cold Water Temperature	16 °C	Cold Water Pressure	3 bar
Hot Water Temperature	62 °C	Hot Water Pressure	3 bar
Room Temperature	18 °C		
Cold Water Filling Time (high level)	45 sec	Voltage	230V ~ 50-60Hz
Hot Water Filling Time (high level)	45 sec		400V 3N ~ 50-60Hz
Mix Water Filling Time (high level)	25 sec		
Disinfection Temperature	80/85/93 °C	Max Temperature Reached	94 °C

## Program 1


Total Time 4' 20"

Water Consumption 18,5 litres

Power Consumption 0,23 kWh

(max 10,7A )

Ao≈155

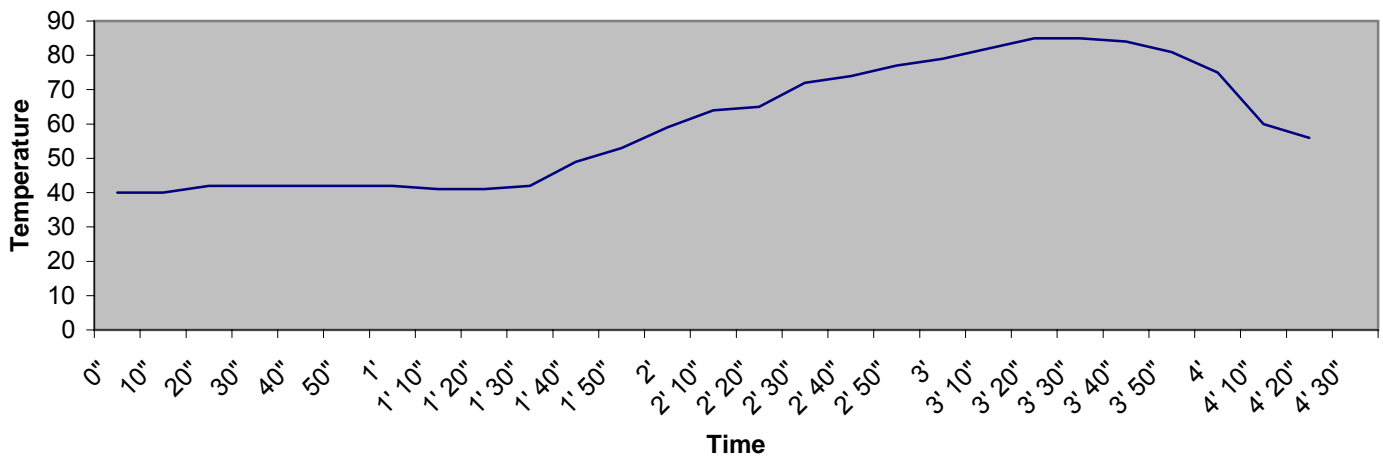

## Program 2


Total Time 5' 24"

Water Consumption 28,9 litres

Power Consumption 0,24 kWh

(max 10,7A )

Ao≈155

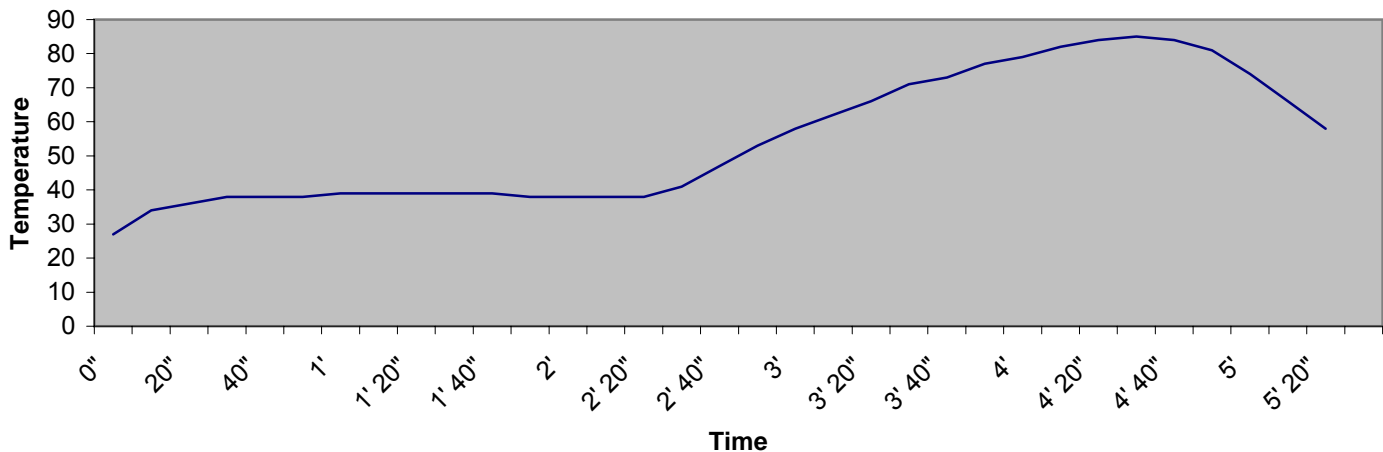

# Ipsos Norge AS.

Trollåsveien 6, 1414 TROLLÅSEN

Tel. 66 99 50 90 - E-mail: post@ipso.no – www.ipso.no

Standard operating conditions: **DVGW03-MULTIW-LE-537F1-MIX**

## Program 3


Total Time 5' 32"

Water Consumption 37 litres

Power Consumption 0,24 kWh

(max 10,7A )

Ao≈155

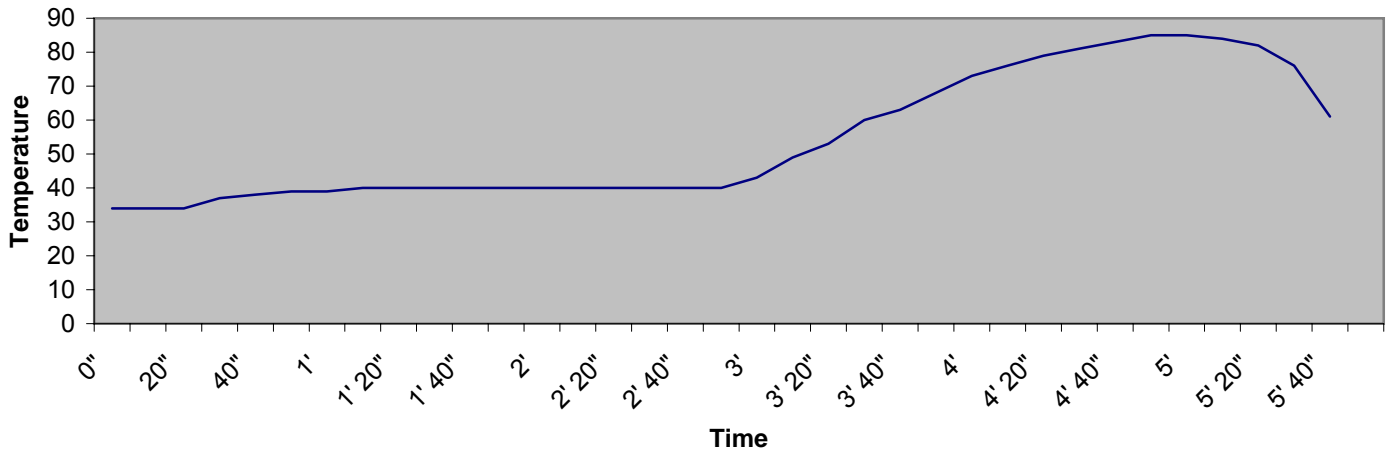

## Program 4

Total Time 9' 02"

Water Consumption 50 litres

Power Consumption 0,39 kWh

(max 10,7A )

Ao≈1400

